

Name

MEDICAL / SURGICAL FACE MASK Type IIR

Article number

MS-IIR

Product data

Type of Product	Type IIR
Colour	Blue
Sterility	Non-sterile
Layer	3
Description	175 x 95 mm, Elastic Earloop, adjustable nosebridge, three layers

Packing

Unit	50 pcs/box
Amount/Carton	N/A
Carton size	N/A

Applicable Standards

EN 14683:2019+AC:2019

EU Directive 93/42/EEC on Medical Devices (MDD),
Annex IX: Class I devices.**Storage Suggestion**

Should be stored in a clean, dry, ventilated environment, related humidity below 80%

Expire Date

5 years after production date.

Cautions

Protects against the spread of particles and droplets.

Do not use for medical or surgical activities where there is a high risk of infection.

If package or product is damaged, prohibit to use!

If reused performance of the product may deteriorate, cross-contamination may occur!

48 box/kartong, 24krt/pall = 57 600st/pall

Art nr: 184897


Performance: (EN 14683:2019+AC:2019)

- Bacterial Filtration Efficiency (BFE): 98,85 %
- Differential pressure: 23 Pa/cm²
- Microbial Cleanliness: 22 cfu/g
- Splash resistance pressure: 16 kPa

Product Description

- Soft and pleasant to skin
- High comfort and easy to breath in
- Type IIR classification



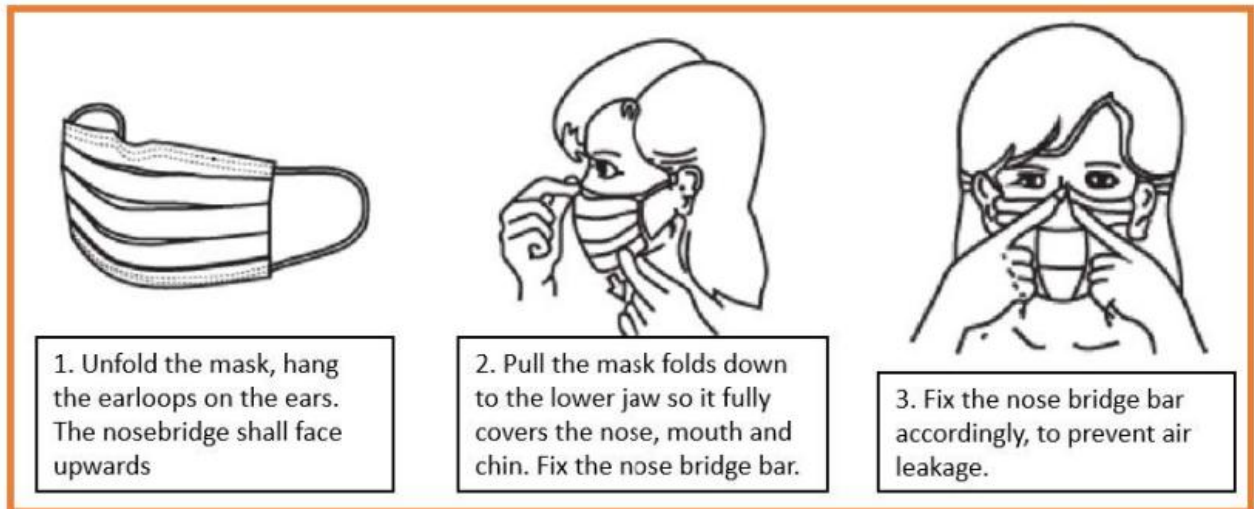

How to put on a face mask

1. Clean your hands with soap and water or hand sanitizer before touching the mask.
2. Remove a mask from the box and make sure there are no obvious tears or holes in either side of the mask.
3. Determine which side of the mask is the top. The side of the mask that has a stiff bendable edge is the top and is meant to mold to the shape of your nose.
4. Determine which side of the mask is the front. The coloured side of the mask is usually the front and should face away from you, while the white side touches your face.
5. Face Mask with Ear loops: Hold the mask by the ear loops. Place a loop around each ear.
6. Pull the bottom of the mask over your mouth and chin.
7. Mold or pinch the stiff edge to the shape of your nose.

How to remove a face mask

1. Clean your hands with soap and water or hand sanitizer before touching the mask. Avoid touching the front of the mask. The front of the mask is contaminated. Only touch the ear loops/ties/band. Follow the instructions below for the type of mask you are using.
2. Face Mask with Ear loops: Hold both ear loops and gently lift and remove the mask
3. Throw the mask in the trash. Clean your hands with soap and water or hand sanitizer

Warnings

- Do not touch the outside of the facemask. If you do, wash your hands properly.
- This mask is only single use. Do not reuse.
- The mask should be replaced in time. Long-term use is not recommendable.
- It is suggested to stop using it if there is any unfitness or adverse reactions in the process of wearing
- The product is not washable, please ensure that it is used within the valid period.
- Store in a dry, ventilated place and stay away from fire and flammable materials
- Change mask if wet, heavy breathable, dirty or at least within (4) hours

STUNDAB
STÅD & UNDERHÅLLS PRODUKTER

www.stundab.se info@stundab.se

Fax. +46 (0)8 54900887 Tel. +46 (0)8 54900888